

Instruction Manual of Disposable Medical Face Mask

[Product name] Disposable medical face mask

[Model] Type A: Ear-loop type

[Specification] Large size: 17.5cm*9.5cm

[Product structure and Raw materials] Disposable medical face mask is composed of mask body, mask band and nose clip. The mask body has three layers of structure: the inner and outer layers are made of non-woven fabric, and the middle layer is made of melt-blown fabric. The nose clip is made of flexible materials. The mask band is made of elastic fabric.

[Certification standards] EN 14683:2019+AC:2019

[Scope of application] The product can be worn by all kinds of clinical personnel during non-invasive operation to cover the mouth, nose and lower jaw of the user so as to provide a certain physical barrier to prevent the direct penetration of pathogenic microorganisms and particles.

[How to use] 1. Before use, check whether the packaging is in good condition and confirm the expiry date of the product.

2. Open the packaging, take out the mask, extend and pull the mask to the face with hands and keep the metallic strip uppermost and the side with folds downward and outward; press the metallic strip from the nose bridge to two sides with fingertips to mould the metallic strip over the nose bridge and the mask over the face.



[Notes] This product is disposable and it is necessary to check the integrity of packaging before use.

[Caution] It is strictly prohibited to use it if the packaging is damaged or the product is expired or reuse it.

[Transportation and storage] Store the product in a dry and ventilated place without corrosive gas.

[Period of validity] One year.

[Batch Number] See the certificate of qualification

[Date of Production] See the certificate of qualification

[Expiry date] See the certificate of qualification

 Henan Chaoya Medical Equipment Co., Ltd.

[Address] No. 19, 13th Street, Zhengzhou Economic and Technological Development Zone, Henan Province, China










[Tel] +86-0371-56571917

EC REP: Luxus Lebenswelt GmbH

[Address] Kochstr. 1, 47877, Willich, Germany

[Tel] 0049-1715605732 **[E-mail]** info_m@luxuslw.de

[Description of graphics, symbols, abbreviations and other contents used in packaging and labels]

| | | | | | | | | |
|---|---|---|---|---|---|--|---|---|
|  |  |  |  |  |  |  |  |  |
| CE MARKING | KEEP AWAY FROM SUNLIGHT | KEEP DRY | RECYCLE | EC REP | MANUFACTURED IN CHINA | UP | DO NOT EXPOSE TO OPEN FLAMES | HANDLE WITH CARE |

[Revision date of manual] April 2020